

# PROLIGNER

Simply Moving Smiles

1. Check impressions meet the required standard
2. Include upper and lower impressions
3. Include bite registration
4. Rinse impressions & bite registration
5. Place impressions in a ziplock bag along with the patient's name
6. Post to the appropriate address

## UK LAB

Ceroplast / Proligner  
11 Bromyard Terrace  
Worcester  
United Kingdom  
WR2 5BW

## AUS LAB


Dentacast / Proligner  
33 Iolanthe St  
Campbelltown  
NSW  
2560

### Bad Impression

Gumline isn't present throughout

Putty isn't mixed properly

Uneven pressure applied from front to back

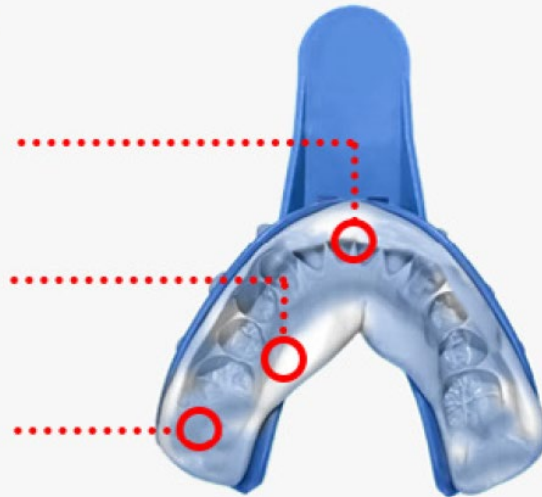

### Good Impression

Single, clear model of your teeth

Gumline Present all the way through

Back molars are present

